ENGLISH Instructions For Use

The Unitek™ Elastomeric Ligature With Guard is designed for use with ceramic brackets to protect opposing occlusal surfaces from potential enamel wear.

The modules are a combination ligature (0.124 in, 3.1 mm outer diameter) and guard so it is not necessary to use a secondary ligature to hold the archwire in place. The modules are provided on convenient disc dispensers suitable for single patient use.

To ligate modules, grip guard portion module with forceps. (Figure 1.)

To ensure proper positioning, start the ligation from the gingival portion of the bracket (Figure 2) and stretch across other wings to ensure bracket engagement. (Figure 3.)

Be sure guard is properly positioned on the occlusal portion of the bracket. (Figure 4.)

These modules may be used in cases where lower arch brackets will occlude with maxillary teeth (i.e., deep bite cases, long cusp tips, finishing, etc.).

Proper use of these modules will reduce the risk of iatrogenic tooth wear caused by an opposing ceramic bracket. (Figure 5.)

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© 3M 2004, 2010, 2013, 2017, 2018, 2020. All rights reserved.